

Asep-RC

Chlorhexidine Gluconate Dental Cleansing Solution

Root Canal Disinfectant Solution

PRODUCT DESCRIPTION

Asep-RC is a Chlorhexidine Gluconate Solution meant for disinfecting the root canal. Asep-RC provides broad spectrum antimicrobial action. Chlorhexidine Gluconate Solution is a potent antimicrobial agent and is therefore ideal for use in irrigation and disinfection of the root canal. Its antifungal property is effective on both vegetative and spore forms.

INTENDED USE / INDICATIONS FOR USE

1. For the disinfection of infected root-canal
2. For rinsing and cleansing the tooth cavity before filling
3. For disinfecting in periodontal diseases

CONTRAINDICATIONS

Asep-RC should not be used in patients who are known to be hyper sensitive to chlorhexidine gluconate or any other ingredients of the formulation. Chlorhexidine Gluconate may cause some tooth discoloration, or increase in tartar formation particularly in areas where stain are tartar usually form.

DIRECTIONS FOR USE

Ream and rinse root canal. Irrigate the root canal, taking 2 ml or adequate quantity of Asep-RC.

Asep-RC is strictly to be used by Dental Professionals only

PRECAUTIONS

1. Do not mix with Sodium Hypochlorite Solution as some precipitation is observed if mixed or used in same syringes
2. Use exclusive syringe for Asep-RC
3. Do not allow Asep-RC to come in contact with any other irrigant
4. Keep the container tightly closed after use

STORAGE

Do not store above 30°C.

SHELF LIFE







24 months




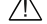

PACKING

150 ml bottles packed in an outer carton with a measuring cup and pack insert.

500 ml bottles packed in an outer carton with a measuring cup dispenser and pack insert.

SYMBOLS

	Medical Device
	Lot No.
	Instruction for Use
	Storage Temperature
	Keep away from sunlight
	Do not use if package is damaged

	Manufactured by
	Date of Manufacture
	Date of Expiry
	Caution
	Flammable

Document No : MD/QA/CLI/CHG-01

Version / Date : 02 / June 04, 2024



Stedman Pharmaceuticals Pvt. Ltd.

A-3, SIDCO Pharmaceutical Complex,
Alathur, Thiruporur - 603 110. Kanchipuram,
Tamilnadu, India.

Marketed by:

Anabond Stedman

Pharma Research (P) Ltd

No. 14A, Ground Floor, 3rd Cross Street, Nehru Nagar,
Kottivakkam, Chennai - 600 041, Tamilnadu

www.anabondstedman.com

For feedback / queries

Email: info@anabondstedman.com

Mobile: +91 73059 65575

DP00248

