



Anabond Stedman

# ANABOND STEDMAN PHARMA RESEARCH

<b>PRODUCT NAME</b>	AS0075 Mucosafe LC	<b>PM CODE</b>	AS0075
<b>DOMESTIC / EXPORT</b>	Domestic	<b>DATE</b>	13.01.2026
<b>SPECIFICATIONS</b>		<b>SOFTWARE</b>	Coreldraw 20
<b>GSM / Type</b>	90 GSM Art Paper		
<b>SIZE</b>	L 135 x H 190 mm	<b>ITEM TYPE</b>	Insert
<b>COLOUR</b>	■ Black		
<b>PACK</b>			
<b>LANGUAGE</b>	English		
<b>CHANGES MADE</b>	New Artwork		
<b>Prepared by:</b>	<b>Checked by:</b>	<b>Approved by:</b>	

# 135 x 190 mm

FOR THE USE OF DENTAL PROFESSIONALS

## MucoSafe<sup>TM</sup> LC

### Temporary Mucosal Protector

#### PRODUCT DESCRIPTION

**MucoSafe LC** is a soft light curable material applied at the protruded part of orthodontic appliance such as orthodontic bracket, orthodontic wire, micro implant and orthodontic dental devices which are used during the orthodontic treatment to reduce the discomfort and mucosal irritation raised by protruded surfaces of orthodontic devices. **MucoSafe LC** is cured at a wavelength of 400-500 nm (blue light) and an intensity of 500 mW/cm<sup>2</sup> for 20 seconds.

#### INTENDED USE / INDICATIONS FOR USE

To protect the mucosa from ulceration caused by protruded surfaces of orthodontic appliances.

#### CONTRAINDICATIONS

**MucoSafe LC** contains acrylates which may cause allergic reactions in some percentage of population.

#### DIRECTIONS FOR USE

- The ideal temperature for optimum handling property is between 18°C and 24°C. If refrigerated, make sure to bring the product to the above mentioned temperature atleast one hour before use.
- Read the pack insert thoroughly before application.
- Open the syringe cap and attach 19G cannula.
- Identify the protruding area to be covered.
- As **MucoSafe LC** is hydrophobic, ensure the surface is thoroughly dried/desiccated.
- Apply **MucoSafe LC** onto the dried/desiccated surface and light cure for 20 seconds using blue light (wavelength 400–500 nm) at an intensity of 500 mW/cm<sup>2</sup>.
- The viscosity and flowability of the product results in soft and smooth finish, without edged parts.

#### PRECAUTIONS

- It is advisable to wear an eyewear while using **MucoSafe LC**.
- MucoSafe LC** contains acrylates which are found to cause allergic reactions in small percentage of population. To reduce the risk of allergic response, minimize exposure to uncured resins.
- Use of protective gloves and no-touch technique is recommended. Acrylates may penetrate commonly used gloves.
- Avoid contact with oral mucosa, eyes and skin. If accidental contact or irritation occurs, flush immediately with sufficient amount of water. In case of eye contact or irritation, please consult a physician.

#### SHELF LIFE

When stored between 10°C and 30°C in the original packages, this product has a shelf life of 24 months from the month of manufacturing.

#### STORAGE

- MucoSafe LC** to be stored between 10°C and 30°C. Storage below or more than the temperature specified, has impact on the product properties.
- Material removed from syringe may be contaminated during use. Do not refill the product to the syringe.
- Make sure that **MucoSafe LC** is always kept closed when not in use.
- Do not store **MucoSafe LC** near eugenol-containing products.







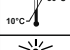
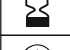



#### PACKING / CONTENT

Each pack contains 1 syringe x 1.5 g (1.2 ml) with 5 cannulas and a pack insert.

#### NOTE

The material is for the use of dental Professionals only. Processing should be carried out strictly according to the instructions for use. Liability cannot be accepted for damages resulting from misuse or failure to observe the instructions including storage conditions. The user is responsible for testing the product for its suitability and use for any other purpose that is not explicitly stated in the instructions.

#### SYMBOLS

 MD	Medical Device		Caution
 LOT	Lot No.		Manufactured by
 i	Instruction for use		Date of Manufacture
 10°C - 30°C	Storage Temperature		Date of Expiry
	Keep away from sunlight		Do not use if package is damaged
			Non Sterile

Version / Date: 00 / January, 2026

  
**Anabond Stedman**  
**Pharma Research Pvt. Ltd.**  
No.117, Kattur Village, Vembudu Panchayat,  
Thirupozur Taluk, Chengalpattu District - 603 108,  
Tamil Nadu

ISO 13485:2016 Certified Company

[www.anabondstedman.com](http://www.anabondstedman.com)

For feedback / queries  
Email: [info@anabondstedman.com](mailto:info@anabondstedman.com)  
Mobile: +91 73059 65675

TM - Trademark under Registration

AS0075

  
**Anabond Stedman**